www.anthos.com • anthos@anthos.it

PROTOCOLLO PER LA MANUTENZIONE IGIENICA DEL RIUNITO
PROTOCOL FOR ENSURING PROPER HYGIENE STANDARDS ON DENTAL UNITS
ПРОТОКОЛ САНИТАРНО-ГИГИЕНИЧЕСКОГО ОБСЛУЖИВАНИЯ УСТАНОВКИ
DIŞ ÜNİTESİNİN HİJYENİK BAKIM PROTOKOLÜ

REF. 97050444 REV. 10

QUANDO • WHEN • КОГДА • NE ZAMAN

IT Dopo ogni paziente
EN After each patient
RU После каждого пациента
TR Her hastadan sonra

IT Ogni mattina
EN Every morning
RU Каждое утро
TR Her sabah

IT Alla fine di ogni giornata di lavoro
EN At the end of the day
RU В конце рабочего дня
TR Her iş günü sonunda

IT Ogni 7 giorni
EN Every 7 days
RU Каждые 7 дней
TR Her 7 günde bir

IT Ogni 14 giorni
EN Every 14 days
RU Каждые 14 дней
TR Her 14 günde bir

IT Una volta al mese
EN Every month
RU Один раз в месяц
TR Ayda bir defa

IT Una volta l'anno
EN Once a year
RU Один раз в год
TR Yılda bir defa

IT Controllare/ pulire se necessario
EN When necessary
RU Проверить/Чистить, если необходимо
TR Gerekmesi halinde kontrol ediniz/ temizleyiniz

IT In caso di soste prolungate (superiori ai 3 giorni)
EN In the event of prolonged standstill (longer than 3 days)
RU При длительном бездействии (более 3 дней)
TR Uzun atılık süreleri durumunda (3 günden fazla)

COME • HOW • KAK • NASIL

IT Indossare mascherina e occhiali
EN Put on mask and safety goggles
RU Пользоваться маской и очками
TR Maske ve gözlük takınız

IT Indossare guanti
EN Put on safety gloves
RU Пользоваться перчатками
TR Eldiven takınız

IT Il contenuto deve essere smaltito fra i rifiuti speciali secondo le Normative Locali e Nazionali
EN Dispose of as hazardous waste, observing current local and national laws
RU Утилизировать содержимое как специальные отходы в соответствии с местными и национальными нормами
TR İçerik, Yerel ve Ulusal Yönetmeliklere uygun olarak, özel atıklar ile imha edilmelidir

IT Sterilizzare in autoclave
EN Part sterilized in autoclave
RU Стерилизовать в автоклаве
TR Otoklavda sterilize ediniz

IT Non spruzzare direttamente sull'apparecchio
EN Do not spray directly onto the unit
RU Не распылять непосредственно на оборудование
TR Direkt olarak cihaz üzerine püskürtmeyiniz

IT Pulire con carta monouso inumidita di disinfettante adatto secondo le istruzioni
EN Clean with disposable paper wetted with a suitable disinfectant as for the instructions
RU Чистить одноразовой бумагой, смоченной в дезинфицирующем средстве в соответствии с инструкциями
TR Tavsiyelere göre, uygun dezenfekte edici ile ıslatılmış tek kullanımlık kağıt ile temizleyiniz

IT Utilizzare disinfettante adatto
EN Use a suitable disinfectant
RU Использовать подходящее дезинфицирующее средство
TR Uygun dezenfekte edici kullanınız

IT Pulire con carta monouso
EN Clean with disposable paper
RU Чистить одноразовой бумагой
TR Tek kullanımlık kağıt ile temizleyiniz

IT Lubrificare
EN Lubricate
RU Смазать
TR Yağlayınız

ITALIANO • ENGLISH • РУССКИЙ • TÜRKÇE

Aspirazione • Suction • Аспирация • Emme	2
Bacinella • Cuspidor Bowl • Ванночка-Плевательница • Tekne.....	4
Strumenti • Instruments • Инструменты • Cihazlar	5
Sistemi di Igiene • Hygiene Systems • Системы Гигиены • Hijyen Sistemleri	8
Superfici • Surfaces • Поверхности • Yüzeyler.....	10

1 FILTRO ASPIRAZIONE • SUCTION FILTER • АСПИРАЦИОННЫЙ ФИЛЬТР • EMME FILTRESİ

IT Una volta svuotato il filtro, sciacquarlo sotto acqua corrente.
EN Once the filter has been emptied rinse it under running water.
RU Ополоснуть фильтр и промыть его в проточной воде.
TR Filtreyi boşalttıktan sonra, tazyikli su altında durulayınız.

IT Sostituire il filtro.
EN Replace filter.
RU Заменить фильтр.
TR Filtreyi değiştiriniz.

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EN Clean and sanitize the suction system using a suitable product approved by ANTHOS. It is recommended to use Ster 3 Plus (Code 97190005). **USE THE CHOSEN PRODUCT IN THE CORRECT DILUTION:** for Ster 3 Plus, 60 ml (1 dosing cup if purchased package) in one litre of hot water (40°C). Prepare the solution in a container and aspirate half the dose for each terminal: immerse the terminal in the solution (for about 2 seconds) and hold it up in the air until the tube has emptied out. Carry out the operation several times until all the solution has been aspirated.

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EN With ACVS (automatic suction flush) execute a flushing cycle using a suitable product approved by ANTHOS. It is recommended to use Ster 3 Plus (Code 97190005). **USE THE CHOSEN PRODUCT IN THE CORRECT DILUTION:** for Ster 3 Plus, 60 ml (1 dosing cup if purchased package) in one litre of water. In the absence of ACVS, aspirate a glass of clean water.

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EN Sterilize the cannula holder terminals in an autoclave after flushing the tubes. The terminals can be subjected to approximately 200 cycles in an autoclave.

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EN Clean the level probes and the vessel with non-abrasive disposable paper. To avoid foam formation in the separator, it is advisable to use anti-foam tablets.

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EN Clean the filter. Once emptied rinse it under running water.

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EN Clean the level probes and the centrifuge probe with non-abrasive disposable paper.

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EN Empty the amalgam container when the device indicates that it is full.

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EN Clean and sanitize the valve by pouring a suitable ANTHOS-approved product inside bowl. It is recommended to use Ster 3 Plus (Code 97190005). **USE THE CHOSEN PRODUCT IN THE CORRECT DILUTION:** for Ster 3 Plus, 60 ml in one litre of hot water (40 °C).

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EN Thoroughly wash at the end of the day. When necessary, use a specific product, for example MD550 Orotol Durr.

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EN Lubricate the O-rings with silicon grease. It is recommended to use S1 (code 91120198).

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EN Clean and disinfect the handpiece with an ANTHOS-approved product. It is recommended to use Ster 1 Plus (Code 97190003). **DO NOT SPRAY ON THE HANDPIECE! ONLY** spout and grip can be sterilized in autoclave. In the event of incorrect spray, clean the nozzle or replace it.

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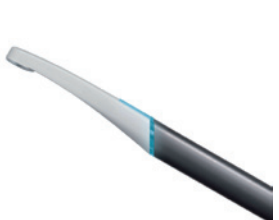
EN Clean any composite residue off the optic fibre with a plastic spatula. **Sterilise the optic fibre ONLY in an autoclave.** Clean and disinfect the handpiece with an ANTHOS-approved product. It is recommended to use Ster 1 plus (code 97190003). **DO NOT SPRAY ON THE HANDPIECE!**

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EN Clean and disinfect both handpiece and Macro-cap with an ANTHOS-approved product. It is recommended to use Ster 1 plus (Code 97190003). **DO NOT SPRAY ON THE HANDPIECE!**

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EN Do not spray directly onto the unit! Do not use abrasive products!

Clean using disposable paper soaked in a disinfecting product, leave it on and dry. Use an ANTHOS-approved product. It is recommended to use Ster 1 Plus (code 97190003). The incorrect use or the use of non-approved products could damage the equipment. In case of stubborn dirt on upholstery, use neutral soap and rinse.

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EN We recommend using the suitable barrier protections for dental unit keyboards.

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EN WARNING: only clean the lamp when it is at room temperature. Frontal shield and dish: clean with a soft damp cloth and neutral soap. Other surfaces: as per the dental unit surfaces, see above. On the Venus Plus, Venus Plus TML version the handles are removable and can be cold-sterilised. The Venus LED lamp handles can be cold-sterilized or in an autoclave. **Do not use abrasive products.** It is recommended to use Ster 1 Plus (Code 97190003).

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EN DENTAL UNIT AND PATIENT CHAIR

We recommend using Ster 1 Plus (code 9719003), by following the indications specified on the label.

Do not spray the product directly on the surfaces.

DO NOT USE products containing:

- isopropyl alcohol (2-propanol, iso-propanol)
- active chlorine (hypochlorites, chloramines and other active Chlorine generators)
- phenols and phenol derivatives
- ethanol (ethyl alcohol) with a concentration above 30%
- 1-propanol (n-propanol, n-propyl alcohol) with a concentration above 20%
- mixtures of ethanol and 1-propanol with a concentration above 40% as a sum of the two alcohols

WARNING! The recommended product is technically compatible with the device materials. Damages to surfaces and materials due to the use of different products cannot be excluded even if they are not included in the exceptions mentioned above.

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