


Duo, Trio, Quattro, Duo Tandem, Quattro Tandem

EN

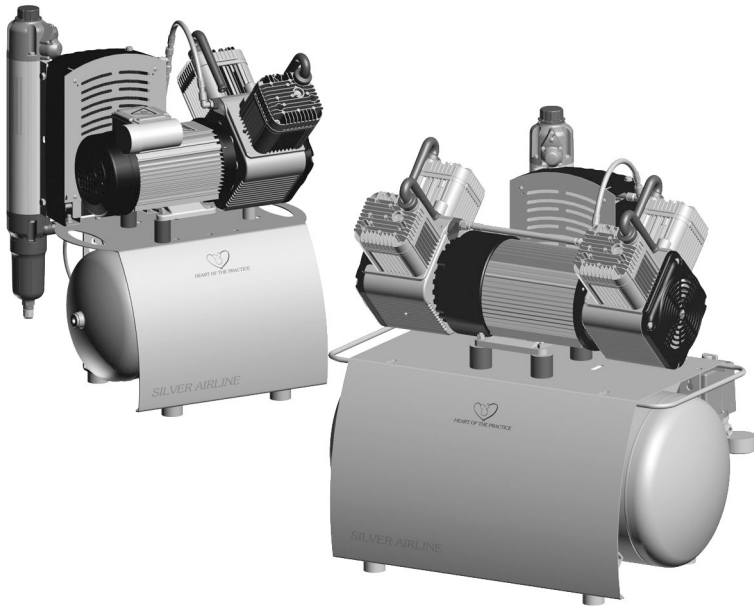

Installation and operating instructions

CE 0297

9000-610-45/30



 DÜRR
DENTAL

1907V008

Contents



Important information

1	About this document	3
1.1	Warnings and symbols	3
1.2	Copyright information	4
2	Safety	4
2.1	Intended purpose	4
2.2	Intended use	4
2.3	Improper use	4
2.4	General safety information	5
2.5	Specialist personnel	5
2.6	Electrical safety	5
2.7	Notification requirement of serious incidents	5
2.8	Only use original parts	5
2.9	Transport	5
2.10	Disposal	6



Product description

3	Overview	7
3.1	Scope of delivery	7
3.2	Optional items	7
3.3	Wear parts and replacement parts	7
4	Technical data	8
4.1	Duo	8
4.2	Duo	10
4.3	Trio	12
4.4	Quattro	14
4.5	Duo Tandem	16
4.6	Quattro Tandem	18
4.7	Distance between rubber feet	20
4.8	Type plate	20
4.9	Evaluation of conformity	20
5	Operation	21
5.1	Duo with membrane drying unit	21



Assembly

6	Requirements	22
6.1	Installation/setup room	22
6.2	Setup	22
6.3	Information about electrical connections	22
7	Transport	23
8	Quattro Tandem assembly and installation	23
9	Installation	24
9.1	Remove the transport locks	24
9.2	Establishing the compressed air connection	24
9.3	Pressure reducer	24
9.4	Place a collector tray underneath	24
9.5	Electrical connections	25
10	Commissioning	25
10.1	Check the motor protection switch	25
10.2	Checking the switch-on/cut-off pressure	26
10.3	Checking the safety valve	26
10.4	Draining the condensation water	27
10.5	Adjusting the rate of flow at the pressure reducer	27
11	Adjustment options	28
11.1	Adjusting the pressure switch	28
11.2	Adjusting the motor protection switch	28
12	Circuit diagrams	30
12.1	1/N/PE AC 110-127 V, 230 V layout	30
12.2	3/N/PE AC 400 V layout	30
12.3	1/N/PE AC 230 V layout, Duo Tandem	31
12.4	3/N/PE AC 230 V layout, Quattro Tandem	32



Usage

EN

13 Operation	33
13.1 Switching the unit on/off	33
14 Maintenance	34
14.1 Maintenance schedule	34
14.2 Replacing the air intake filter ...	35
14.3 Replacing the filter of the mem- brane drying unit	35
15 Taking out of use	36
15.1 Taking the unit out of use	36
15.2 Storage of the unit	36



Troubleshooting

16 Tips for operators and service techni- cians	37
--	----



Appendix

17 Handover protocol	39
-----------------------------------	----

Important information

1 About this document

These installation and operating instructions represent part of the unit.



If the instructions and information in these installation and operating instructions are not followed, Dürr Dental will not be able to offer any warranty or assume any liability for the safe operation and the safe functioning of the unit.

The German version of the installation and operating instructions is the original manual. All other languages are translation of the original manual. These installation and operating instructions apply to:

Duo

Order number: 5252100001; 5252-01; 5252-01/Service; 5252-51

Trio

Order number: 5352-01; 5352100001

Quattro

Order number: 5452-51; 5452100001

Duo Tandem

Order number: 4252-01; 4252100004

Quattro Tandem

Order number: 4682-52; 4682-53

1.1 Warnings and symbols

Warnings

The warnings in this document are intended to draw your attention to possible injury to persons or damage to machinery.

The following warning symbols are used:



General warning symbol



Warning – dangerous high voltage



Warning – hot surfaces



Warning - automatic start-up of the unit

The warnings are structured as follows:



SIGNAL WORD

Description of the type and source of danger

- Here you will find the possible consequences of ignoring the warning
- Follow these measures to avoid the danger.

The signal word differentiates between four levels of danger:

- **DANGER**
Immediate danger of severe injury or death
- **WARNING**
Possible danger of severe injury or death
- **CAUTION**
Risk of minor injuries
- **NOTICE**
Risk of extensive material/property damage

Other symbols

These symbols are used in the document and on or in the unit:



Note, e.g. specific instructions regarding efficient and cost-effective use of the unit.



Observe the operating instructions.



Disconnect all power from the unit.



Refer to the accompanying electronic documents.



Air



Dispose of correctly in accordance with EU Directive 2012/19/EU (WEEE).



CE labelling with the number of the notified body



Order number



Serial number



Medical device



Health Industry Bar Code (HIBC)



1.2 Copyright information

All circuits, processes, names, software programs and units mentioned in this document are protected by copyright.

The Installation and Operating Instructions must not be copied or reprinted, neither in full nor in part, without written authorisation from Dürre Dental.

2 Safety

Dürre Dental has designed and constructed this unit so that when used properly and for the intended purpose it does not pose any danger to people or property.

Despite this, the following residual risks can remain:

- Personal injury due to incorrect use/misuse
- Personal injury due to mechanical effects
- Personal injury due to electric shock
- Personal injury due to radiation
- Personal injury due to fire
- Personal injury due to thermal effects on skin
- Personal injury due to lack of hygiene, e.g. infection



WARNING

The development of emphysema

Soft tissue can be damaged as a result of careless handling.

- › Do not dwell in the area being treated for any longer than is necessary.

2.1 Intended purpose

The compressor is designed to supply compressed air for dental applications.

2.2 Intended use

The air supplied by the compressor is suitable for driving dental tools.

The compressed air generated by the compressor is delivered to the pipeline system of the surgery. The entire compressed air system must be designed in such a way that the quality of the compressed air generated by the compressor is not impaired.

With this prerequisite, the air provided by the compressor is also suitable for blow-drying tooth preparations.

2.3 Improper use

Any use of this appliance / these appliances above and beyond that described in the Installation and Operating Instructions is deemed to be incorrect usage. The manufacturer cannot be held liable for any damage resulting from incorrect usage. The operator will be held liable and bears all risks.

**WARNING****Risk of explosion due to ignition of combustible materials**

- › Do not operate the unit in any rooms in which inflammable mixtures may be present, e.g. in operating theatres.
- › The unit is not suitable for providing an air supply to respirators.
- › This unit is not suitable for drawing up fluids or for compressing aggressive gases or potentially explosive gases.

2.4 General safety information

- › Always comply with the specifications of all guidelines, laws, and other rules and regulations applicable at the site of operation for the operation of this unit.
- › Check the function and condition of the unit prior to every use.
- › Do not convert or modify the unit.
- › Comply with the specifications of the Installation and Operating Instructions.
- › The Installation and Operating Instructions must be accessible to all operators of the unit at all times.

2.5 Specialist personnel**Operation**

Unit operating personnel must ensure safe and correct handling based on their training and knowledge.

- › Instruct or have every user instructed in handling the unit.

The following groups are not permitted to operate or use a commercially operated unit:

- People without the necessary experience and knowledge
- People with reduced physical, sensory or mental capabilities
- Children

Installation and repairs

- › Installation, readjustments, alterations, upgrades and repairs must be carried out by Dürr Dental or by qualified personnel specifically approved and authorized by Dürr Dental.

2.6 Electrical safety

- › Observe and comply with all the relevant electrical safety regulations when working on the unit.
- › Replace any damaged cables or plugs immediately.

2.7 Notification requirement of serious incidents

The user / patient is required to report to the manufacturer and the competent authority of the Member State, in which the user and/or patient is established, any serious incident that has occurred in relation to the device.

2.8 Only use original parts

- › Only use Dürr Dental parts or accessories and special accessories specifically approved by Dürr Dental.
- › Only use only original wear parts and replacement parts.



Dürr Dental accepts no liability for damages or injury resulting from the use of non-approved accessories or optional accessories, or from the use of non-original wear parts or replacement parts.

The use of non-approved accessories, optional accessories or non-genuine wear parts / replacement parts (e.g. mains cable) can have a negative effect in terms of electrical safety and EMC.

2.9 Transport

The original packaging provides optimum protection for the unit during transport.

If required, original packaging for the unit can be ordered from Dürr Dental.



Dürr Dental will not accept any responsibility or liability for damage occurring during transport due to the use of incorrect packaging, even where the unit is still under guarantee.

- › Only transport the unit in its original packaging.
- › Keep the packing materials out of the reach of children.

2.10 Disposal



The unit must be disposed of properly. Within the European Union, the unit must be disposed of in accordance with EU Directive 2012/19/EU (WEEE).

EN

› If you have any questions about the correct disposal of parts, please contact your dental trade supplier.



An overview of the waste keys for Dürr Dental products can be found in the download area at www.duerrdental.com (document no. P007100155).



Product description

3 Overview

3.1 Scope of delivery

The following items are included in the scope of delivery (possible variations due to country-specific requirements and/or import regulations):

<i>Duo 110 V, 1~, with membrane drying unit</i>	5252100001
<i>Duo 230 V, 1~, with membrane drying unit</i>	5252-01
<i>Duo 230 V, 1~, with membrane drying unit</i>	5252-01/Service
<i>Duo 400 V, 3~, with membrane drying unit</i>	5252-51
<i>Trio 230 V, 1~, with membrane drying unit</i>	5352-01
<i>Trio 230 V, 1~, with membrane drying unit</i>	5352100001
<i>Quattro 400 V, 3~, with membrane drying unit *</i>	5452-51
<i>Quattro 400 V, 3~, with membrane drying unit *</i>	5452100001
<i>Duo Tandem 230 V, 1~, with membrane drying unit</i>	4252-01
<i>Duo Tandem CAD/CAM 230 V, 1~, with membrane drying unit *</i>	4252100004
<i>Quattro Tandem 230 V, 3~, with membrane drying unit *</i>	4682-52
<i>Quattro Tandem 400 V, 3~, with membrane drying unit *</i>	4682-53

- Fabric reinforced hose
- Hose nozzle
- Hose clip
- Installation and operating instructions
- Appliance log book
- Collector tray

* with sterile filter

3.2 Optional items

The following items can optionally be used with the unit; these items do not bear the CE mark:

Pressure reducer 6040-992-00

Sterile filter 1640-981-00

Wooden cabinet for for sound insulation of Duo Compressors 5150-500-00

Wooden cabinet for for sound insulation of Duo Tandem, Trio and Quattro Compressors 4251-500-00

3.3 Wear parts and replacement parts

The following working parts must be replaced at regular intervals (refer also to "Maintenance"); these articles do not bear the CE mark:

Air intake filter	0832-982-00
Fine filter	1610-121-00
Sterile filter	1640-981-00
Sintered filter	1650-101-00
Fleece filter	4280-982-00



Any repairs exceeding routine maintenance may only be carried out by qualified personnel or our service.



Information about replacement parts is available from the portal for authorised specialist dealers at: www.duerrdental.net.



If the mains cable of this unit is damaged it must only be replaced by an original mains cable from the manufacturer.

4 Technical data

4.1 Duo

EN

Electrical data		5252-01		5252-51	
		5252-01/Service			
Rated voltage	V	230		400	
Mains frequency	Hz	50	60	50	60
Nominal current at 8 bar (0.8 MPa)	A	6.3	7.0	3.1	2.5
Motor protection switch, recommended settings	A	6.5	7.6	3.1	2.5
Speed	rpm	1360	1600	1410	1690
Type of protection		IP 24		IP 24	
Mains fuses *	A	10		10	
Max. permissible mains impedance in accordance with EN 61000-3-11 **	Ω	≤ 0.142		-	

* Circuit breaker fuse characteristics B, C or D in acc. with EN 60898-1

** Mains impedance at 6 switching cycles per hour. If the number of switching cycles per hour is higher a lower mains impedance is required.

General technical data

Pressure tank volume	l	20		20	
Suction power, approx.	l/min	210	255	210	255
Delivery at 5 bar (0.5 MPa) *	l/min	115	130	115	130
Pressure build-up phase 0 - 7.5 bar (0 - 0.75 MPa) approx.	s	80	70	80	70
Duty cycle	%	100		100	
Start-up pressure	bar (MPa)	6 (0.6)		6 (0.6)	
Cut-off pressure	bar (MPa)	7.8 (0.78)		7.8 (0.78)	
Cut-off pressure, max. adjustable	bar (MPa)	9.5 (0.95)		9.5 (0.95)	
Safety valve, maximum permissible operating pressure	bar (MPa)	10 (1)		10 (1)	
Pressure dew point at 7 bar (0.7 MPa) **	$^{\circ}\text{C}$	$\leq +5$		$\leq +5$	
Dimensions (H x W x D) ***	cm	69 x 49 x 47		69 x 49 x 47	
Weight	kg	50		50	
Noise level ****					
without sound insulation	dB(A)	66	68	66	68
with sound insulation	dB(A)	55	58	55	58

* Delivery without membrane drying unit, at +20 $^{\circ}\text{C}$ and 1013 mbar (0.1 MPa)

** Value determined at an ambient temperature of +40 $^{\circ}\text{C}$

*** Values without accessories and add-on parts

**** Noise level in accordance with ISO 3744

Filter mesh size

Compressor air intake filter	µm	3
Fine filter for membrane drying unit	µm	3
Sterile filter for membrane drying unit	µm	0.01
Sintered filter for membrane drying unit	µm	35

Ambient conditions during storage and transport

Temperature	°C	-10 to +55
Relative humidity	%	max. 95

Ambient conditions during operation

Temperature	°C	+10 to +40
Ideal temperature	°C	+10 to +25
Relative humidity	%	max. 95

Classification

Medical devices class	IIa
-----------------------	-----

4.2 Duo

Electrical data		5252100001	
Rated voltage	V	110-115	110-127
Mains frequency	Hz	50	60
Nominal current at 8 bar (0.8 MPa)	A	14.1-14.6	14.2-13.3
Motor protection switch, recommended settings	A	17	16
Speed	rpm	1350-1370	1560-1640
Type of protection		IP 24	
Mains fuses *	A	20	

* Circuit breaker fuse characteristics B, C or D in acc. with EN 60898-1

General technical data			
Pressure tank volume	l	20	
Suction power, approx.	l/min	210	255
Delivery at 5 bar (0.5 MPa) *	l/min	115	130
Pressure build-up phase 0 - 7.5 bar (0 - 0.75 MPa) approx.	s	80	70
Duty cycle	%	100	
Start-up pressure	bar (MPa)	6 (0.6)	
Cut-off pressure	bar (MPa)	7.8 (0.78)	
Cut-off pressure, max. adjustable	bar (MPa)	9.5 (0.95)	
Safety valve, maximum permissible operating pressure	bar (MPa)	10 (1)	
Pressure dew point at 7 bar (0.7 MPa) **	°C	≤ +5	
Dimensions (H x W x D) ***	cm	69 x 49 x 47	
Weight	kg	50	
Noise level ****			
without sound insulation	dB(A)	66	68
with sound insulation	dB(A)	55	58

* Delivery without membrane drying unit, at +20°C and 1013 mbar (0.1 MPa)

** Value determined at an ambient temperature of +40 °C

*** Values without accessories and add-on parts

**** Noise level in accordance with ISO 3744

Filter mesh size		
Compressor air intake filter	µm	3
Fine filter for membrane drying unit	µm	3
Sterile filter for membrane drying unit	µm	0.01
Sintered filter for membrane drying unit	µm	35

Ambient conditions during storage and transport

Temperature	°C	-10 to +55
Relative humidity	%	max. 95

Ambient conditions during operation

Temperature	°C	+10 to +40
Ideal temperature	°C	+10 to +25
Relative humidity	%	max. 95

Classification

Medical devices class	IIa
-----------------------	-----

4.3 Trio

Electrical data	5352-01 5352100001	
Rated voltage	V	230
Mains frequency	Hz	50
Nominal current at 8 bar (0.8 MPa)	A	8.6
Motor protection switch, recommended settings	A	8.6
Speed	rpm	1350
Type of protection		IP 24
Mains fuses *	A	10
Max. permissible mains impedance in accordance with EN 61000-3-11	Ω	≤ 0.078

* Circuit breaker fuse characteristics B, C or D in acc. with EN 60898-1

General technical data		
Pressure tank volume	l	50
Suction power, approx.	l	315
Delivery at 5 bar (0.5 MPa) *	l/min	160
Pressure build-up phase 0 - 7.5 bar (0 - 0.75 MPa) c.	s	140
Duty cycle	%	100
Start-up pressure	bar (MPa)	6 (0.6)
Cut-off pressure	bar (MPa)	7.8 (0.78)
Cut-off pressure, max. adjustable	bar (MPa)	9.5 (0.95)
Safety valve, maximum permissible operating pressure	bar (MPa)	10 (1)
Pressure dew point at 7 bar (0.7 MPa) **	$^{\circ}\text{C}$	$\leq +5$
Dimensions (H x W x D) ***	cm	76 x 74 x 52
Weight	kg	80
Noise level ****		
without sound insulation	dB(A)	67
with sound insulation	dB(A)	54

* Delivery without membrane drying unit, at +20 $^{\circ}\text{C}$ and 1013 mbar (0.1 MPa)

** Value determined at an ambient temperature of +40 $^{\circ}\text{C}$

*** Values without accessories and add-on parts

**** Noise level in accordance with ISO 3744

Filter mesh size		
Compressor air intake filter	μm	3
Fine filter for membrane drying unit	μm	3
Sterile filter for membrane drying unit	μm	0.01

Filter mesh size

Sintered filter for membrane drying unit	μm	35
--	----	----

Ambient conditions during storage and transport

Temperature	°C	-10 to +55
Relative humidity	%	max. 95

Ambient conditions during operation

Temperature	°C	+10 to +40
Ideal temperature	°C	+10 to +25
Relative humidity	%	max. 95

Classification

Medical devices class		Ila
-----------------------	--	-----

4.4 Quattro

Electrical data		5452-51 5452100001	
Rated voltage	V	400	
Mains frequency	Hz	50	60
Nominal current at 8 bar (0.8 MPa)	A	4.4	4.8
Motor protection switch, recommended settings	A	5.0	5.0
Speed	rpm	1440	1700
Type of protection		IP 24	
Mains fuses *	A	10	
Max. permissible mains impedance in accordance with EN 61000-3-11 **	Ω	≤ 0.24	

* Circuit breaker fuse characteristics B, C or D in acc. with EN 60898-1

** Mains impedance at 6 switching cycles per hour. If the number of switching cycles per hour is higher a lower mains impedance is required.

General technical data			
Pressure tank volume	l	50	
Suction power, approx.	l/min	420	505
Delivery at 5 bar (0.5 MPa) *	l/min	220	255
Pressure build-up phase 0 - 7.5 bar (0 - 0.75 MPa) c.	s	100	90
Duty cycle	%	100	
Start-up pressure	bar (MPa)	6 (0.6)	
Cut-off pressure	bar (MPa)	7.8 (0.78)	
Cut-off pressure, max. adjustable	bar (MPa)	9.5 (0.95)	
Safety valve, maximum permissible operating pressure	bar (MPa)	10 (1)	
Pressure dew point at 7 bar (0.7 MPa) **	$^{\circ}\text{C}$	$\leq +5$	
Dimensions (H x W x D) ***	cm	76 x 74 x 52	
Weight	kg	85	
Noise level ****			
without sound insulation	dB(A)	69	70
with sound insulation	dB(A)	54	58

* Delivery without membrane drying unit, at +20 $^{\circ}\text{C}$ and 1013 mbar (0.1 MPa)

** Value determined at an ambient temperature of +40 $^{\circ}\text{C}$

*** Values without accessories and add-on parts

**** Noise level in accordance with ISO 3744

Filter mesh size		
Compressor air intake filter	μm	3

Filter mesh size

Fine filter for membrane drying unit	µm	3
Sterile filter for membrane drying unit	µm	0.01
Sintered filter for membrane drying unit	µm	35

Ambient conditions during storage and transport

Temperature	°C	-10 to +55
Relative humidity	%	max. 95

Ambient conditions during operation

Temperature	°C	+10 to +40
Ideal temperature	°C	+10 to +25
Relative humidity	%	max. 95

Classification

Medical devices class	IIa
-----------------------	-----

4.5 Duo Tandem

Electrical data		4252-01		4252100004	
Rated voltage	V	230		230	
Mains frequency	Hz	50	60	50	60
Nominal current at 8 bar (0.8 MPa)	A	12.2	14.3	12.2	14.3
Motor protection switch, recommended settings	A	6.5	7.6	6.5	7.6
Speed	rpm	1410	1690	1410	1690
Type of protection		IP 24		IP 24	
Mains fuses *	A	20		20	
Max. permissible mains impedance in accordance with EN 61000-3-11 **	Ω	≤ 0.3		≤ 0.3	

* Circuit breaker fuse characteristics B, C or D in acc. with EN 60898-1

** Mains impedance at 6 switching cycles per hour. If the number of switching cycles per hour is higher a lower mains impedance is required.

General technical data

Pressure tank volume	l	50		50	
Suction power, approx.	l/min	420	505	420	505
Delivery at 5 bar (0.5 MPa) *	l/min	225	260	225	260
Pressure build-up phase 0 - 7.5 bar (0 - 0.75 MPa) c.	s	100	90	100	90
Duty cycle	%	100		100	
Start-up pressure	bar (MPa)	6 (0.6)		> 7 (0.7)	
Cut-off pressure	bar (MPa)	7.8 (0.78)		9 (0.9)	
Cut-off pressure, max. adjustable	bar (MPa)	9.5 (0.95)		9.5 (0.95)	
Safety valve, maximum permissible operating pressure	bar (MPa)	10 (1)		10 (1)	
Pressure dew point at 7 bar (0.7 MPa) **	$^{\circ}\text{C}$	$\leq +5$		$\leq +5$ ***	
Dimensions (H x W x D) ****	cm	76 x 79 x 52		76 x 79 x 52	
Weight	kg	95		95	
Noise level *****					
without sound insulation	dB(A)	69	72	69	72
with sound insulation	dB(A)	55	56	55	56

** Value determined at an ambient temperature of +40 $^{\circ}\text{C}$

*** With orange rinsing nozzle

**** Values without accessories and add-on parts

***** Noise level in accordance with ISO 3744

Filter mesh size

Compressor air intake filter	μm	3			
Sterile filter for membrane drying unit	μm	0.01			

Filter mesh size

Sintered filter for membrane drying unit	μm	35
--	----	----

Ambient conditions during storage and transport

Temperature	°C	-10 to +55
Relative humidity	%	max. 95

Ambient conditions during operation

Temperature	°C	+10 to +40
Ideal temperature	°C	+10 to +25
Relative humidity	%	max. 95

Classification

Medical devices class		IIa
-----------------------	--	-----

4.6 Quattro Tandem

Electrical data		4682-52		4682-53	
Rated voltage	V	230 / 3~		400 / 3~	
Mains frequency	Hz	50	60	50	60
Nominal current at 8 bar (0.8 MPa)	A	15.2	16.6	8.8	9.6
Motor protection switch, recommended settings	A	9	9	5	5
Speed	rpm	1440	1700	1440	1700
Type of protection		IP 24		IP 24	
Mains fuses *	A	20		20	
Max. permissible mains impedance in accordance with EN 61000-3-11 **	Ω	≤ 0.445		≤ 0.18	

* Circuit breaker fuse characteristics B, C or D in acc. with EN 60898-1

** Mains impedance at 6 switching cycles per hour. If the number of switching cycles per hour is higher a lower mains impedance is required.

General technical data

Pressure tank volume	l	90		90	
Suction power, approx.	l/min	845	1010	845	1010
Delivery at 5 bar (0.5 MPa) *	l/min	440	515	440	515
Pressure build-up phase 0 - 7.5 bar (0 - 0.75 MPa) c.	s	90	80	90	80
Duty cycle	%	100		100	
Start-up pressure	bar (MPa)	6.5 (0.65)		6.5 (0.65)	
Cut-off pressure	bar (MPa)	8.5 (0.85)		8.5 (0.85)	
Cut-off pressure, max. adjustable	bar (MPa)	9.5 (0.95)		9.5 (0.95)	
Safety valve, maximum permissible operating pressure	bar (MPa)	10 (1)		10 (1)	
Pressure dew point at 7 bar (0.7 MPa) **	$^{\circ}\text{C}$	$\leq +5$		$\leq +5$	
Dimensions (H x W x D) ***	cm	76 x 102 x 62		76 x 102 x 62	
Weight	kg	170		170	
Noise level ****	dB(A)	72	73	72	73

* Delivery without membrane drying unit, at +20 $^{\circ}\text{C}$ and 1013 mbar (0.1 MPa)

** Value determined at an ambient temperature of +40 $^{\circ}\text{C}$

*** Values without accessories and add-on parts

**** Noise level in accordance with ISO 3744

Filter mesh size

Compressor air intake filter	μm	3
Fine filter for membrane drying unit	μm	3
Sterile filter for membrane drying unit	μm	0.01

Filter mesh size

Sintered filter for membrane drying unit	μm	35
--	----	----

Ambient conditions during storage and transport

Temperature	°C	-10 to +55
Relative humidity	%	max. 95

Ambient conditions during operation


Temperature	°C	+10 to +40
Ideal temperature	°C	+10 to +25
Relative humidity	%	max. 95

Classification

Medical devices class		IIa
-----------------------	--	-----

4.7 Distance between rubber feet

Distances between the rubber feet for different pressure vessel volumes:

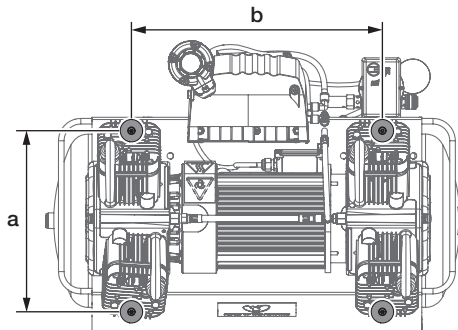


	a (cm)	b (cm)
20 l	23	27
50 l	32.5	45
90 l	32.5	59

4.8 Type plate

Complete system


The type plate of the overall system is located on the mounting plate for the compressor unit.

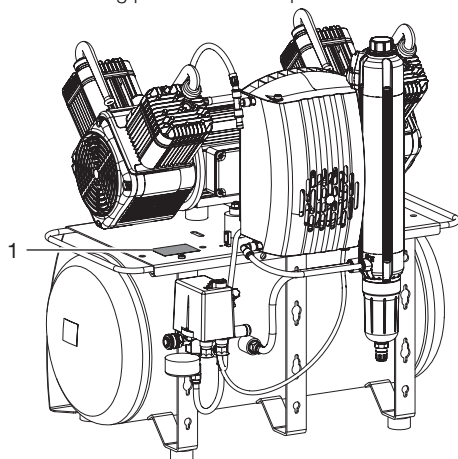

1 Type plate for the complete system

Compressor unit


The type plate of the compressor unit is located on the crankcase below the cylinder.

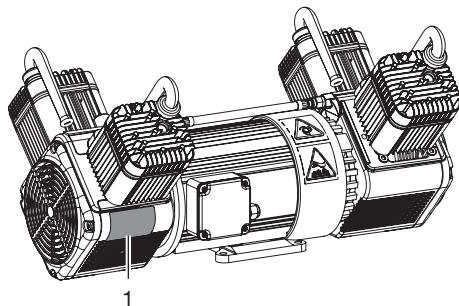

1 Compressor unit type plate

Membrane drying unit

The type plate of the membrane drying unit is located on the side of the membrane drying unit.

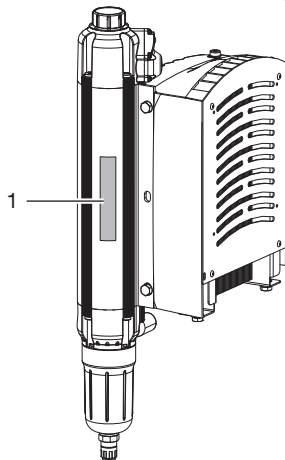

1 Membrane drying unit type plate


4.9 Evaluation of conformity

This device has been subjected to conformity acceptance testing in accordance with the current relevant European Union guidelines. This equipment conforms to all relevant requirements.

5 Operation

5.1 Duo with membrane drying unit

EN

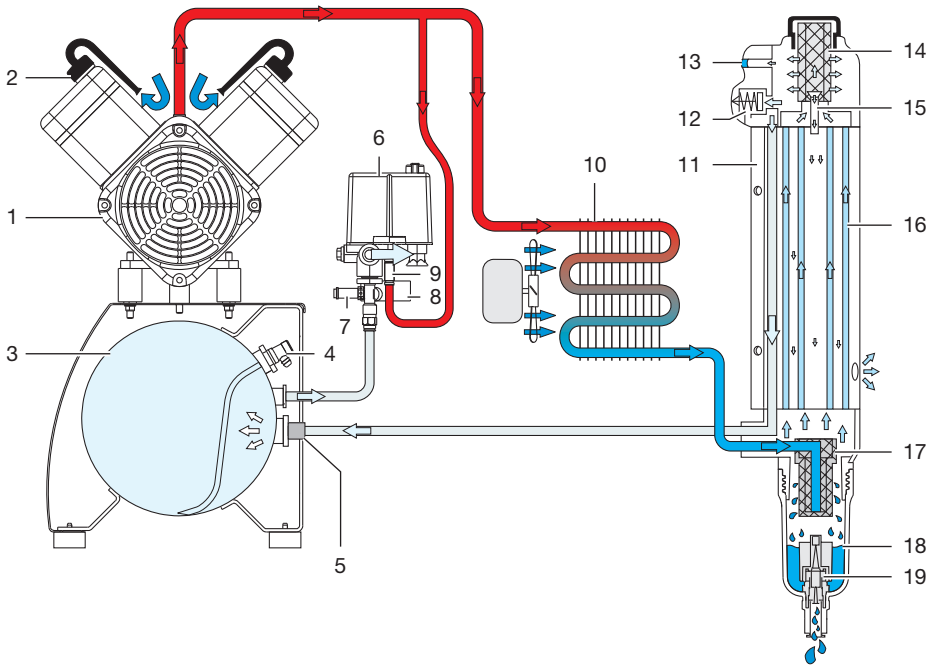

- 1 Compressor unit
- 2 Air intake filter
- 3 Pressure tank
- 4 Condensate drain valve
- 5 Non-return valve
- 6 Pressure switch
- 7 Safety valve
- 8 Pressure gauge/display
- 9 Pressure relief valve
- 10 Cooler with ventilating fan
- 11 Membrane drying unit
- 12 Pressure limiting valve
- 13 Humidity display
- 14 Fine or sterile filter
- 15 Rinsing nozzle
- 16 Membrane fibre
- 17 Sinter filter
- 18 Water collection chamber
- 19 Water outlet valve

The compressor unit draws in atmospheric air and compresses it without oil. It then transports the oil-free compressed air to the membrane drying unit. The cooler and the membrane dryer extract moisture from the compressed air. The oil-free, hygienic and dry air is stored in the pressure tank ready for use in connected devices.

Assembly

EN

6 Requirements



The unit must not be set up or operated within the vicinity of the patients (within a radius of 1.5 m).

The unit can be installed either at the same level as the surgery room or on a floor below (e.g. cellar).

Due of the amount of noise generated, we recommend that the unit is installed in an adjoining room.

The pipes provided on-site must at least meet the country-specific requirements for drinking water.



Further information can be found in our separate planning information leaflet for compressed air.

6.1 Installation/setup room

The room chosen for set up must fulfil the following requirements:

- Closed, dry, well-ventilated room
- Should not be a room made for another purpose (e. g. boiler room or wet cell)
- If the unit is installed in a machine room, e.g. in an adjoining room or cellar, the requirements set out in ISO-TS 22595 must be complied with.




ACHTUNG

Risk of overheating due to insufficient ventilation

The units generates heat. Possibility of heat damage and/or reduced service life of the unit.

- › Do not cover the unit.
- › Install a fan for auxiliary ventilation in rooms where ambient temperatures exceed ≥ 40 °C while the unit is in operation.

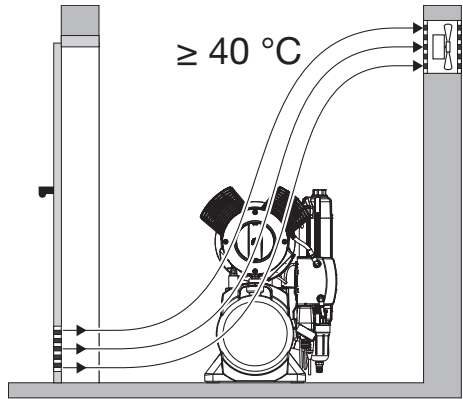

6.2 Setup

The following conditions must be taken into account for installation:



The air is filtered when it is sucked in. This does not alter the composition of the air. For this reason it is important to keep the sucked-in air free of harmful substances (e.g. do not suck in exhaust gases or contaminated exhaust air).

- Clean, level and sufficiently stable subsurface (note the weight of the unit).
- Type plate easy to read.
- Unit easy to access for operation and maintenance.
- Easy-to-access power outlet to which the unit is connected.
- Maintain sufficient distance to the wall (at least 20 cm).
- The compressed air pipe should be routed as closely as possible to the place of installation (note the length of the hose supplied).

6.3 Information about electrical connections

- › Ensure that the electrical connections to the mains power supply are established in accordance with current valid national and local regulations and standards governing the installation of low voltage units in medical facilities.
- › Observe the current consumption of the devices that are to be connected.

7 Transport



WARNING


Risk of explosion of the pressure tank and pressure hoses

- › The pressure tank and the pressure hoses must be vented before they are stored or transported.
- › Protect the unit against moisture, dirt and extreme temperatures during transport ("4 Technical data").
- › Always make sure that the condensate collector chamber is empty before transporting the unit ("15 Taking out of use").
- › Always transport the unit in an upright position.
- › Only transport the unit using the transport handles provided.
- › Check the unit for transport damage.


8 Quattro Tandem assembly and installation

For weight reasons the unit is not supplied fully assembled. Instead, the compressor units are installed at the site of use.

- › Set up the tank at the planned installation site.
- › Screw in the vibration dampers into the motor mounting.
- › Place the compressors on the vibration dampers.
- › Attach the compressor units with the lock washers and nuts.

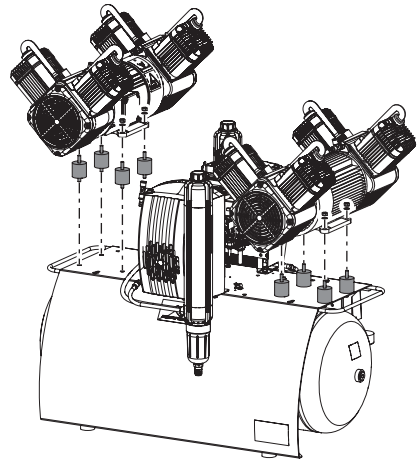

- › Insert the compressed air connections from the compressor unit into the cooler.
- › Plug in the electrical connections of the compressor units at the control box. Connect the left-hand unit to the left-hand plug connector and the right-hand unit to the right-hand plug connector.

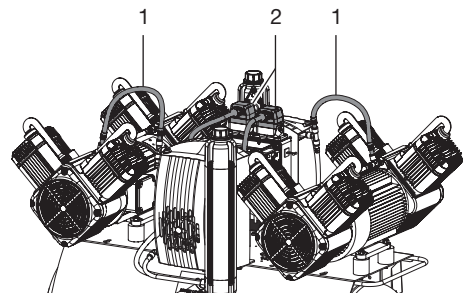

- 1 Compressed air connection
- 2 Electrical connection

9 Installation

9.1 Remove the transport locks

For safe transport, the unit is secured with foam padding blocks and a transport strap.

- › Cut and remove the transport strap.
- › Remove the foam padding blocks.

9.2 Establishing the compressed air connection



The supplied flexible pressure hose between the pipe system and the compressor prevents vibrations from being transmitted and thus reduces noise. This ensures safe and reliable operation.

- › Connect the pre-assembled connecting sleeve of the pressure hose to the quick release coupling.
- › Measure the required length of the pressure hose and shorten if necessary.
- › Press a fitting hose connector (not included in the scope of delivery) onto the pressure hose (internal diameter 10 mm) and secure it with a hose clip.
- › Connect the connecting sleeve of the pressure hose to the compressed air tube.



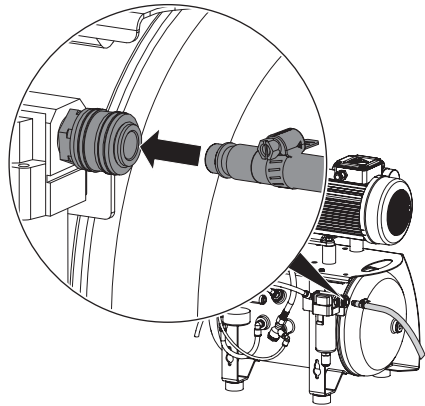


Fig. 1: Duo with condensate separator

9.3 Pressure reducer

- › Insert the pressure reducer into the quick release coupling.
- › Insert the pressure hose into the quick release coupling on the pressure reducer.

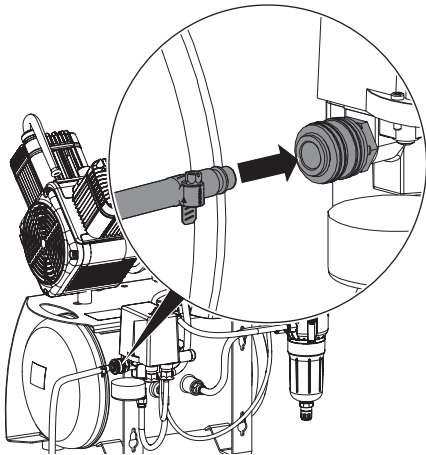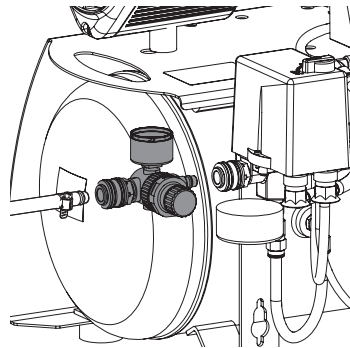


9.4 Place a collector tray underneath

During operation, condensation water on the unit is continuously separated and automatically drained. In order to prevent water damage due to drained condensation, it is collected in the collector tray.




As an option, the condensation can also be drained through a hose into the waste water system. Always comply with applicable national regulations for waste water systems.

- › Place a collector tray under the condensate separator or the membrane drying unit (depending on type).

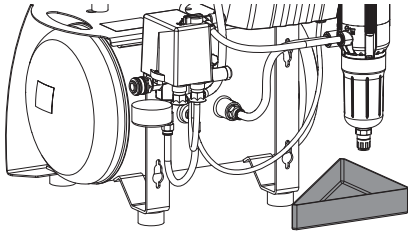

9.5 Electrical connections

Safety when making electrical connections

-  The unit has no main power switch. For this reason it is important that the unit is set up in such a way that the plug can be easily accessed and unplugged if required.
- › The unit must only be connected to a correctly installed power outlet.
- › Make sure that none of the electrical cables leading to the unit are under any mechanical tension.
- › Before initial start-up check that the mains supply voltage and the voltage stated on the type plate match (see also "4. Technical data").

Establishing the electrical connections





DANGER

Risk of electric shock due to defective mains cable

- › Mains cables must not be allowed to come into contact with any hot surfaces on the unit.
- › Connect the mains plug to an earthed power outlet.


10 Commissioning

-  In many countries technical medical products and electrical devices are subject to regular checks at set intervals. The owner must be instructed accordingly.
- › Turn on the unit power switch or the main surgery switch.
- › Carry out an electrical safety check in accordance with applicable local regulations (e.g. the German Ordinance on the Installation, Operation and Use of Medical Devices / Medizinprodukte-Betreiberverordnung) and record the results as appropriate, e.g. in the technical log book.
- › Carry out and document the instruction and handover for the unit.
-  A sample handover report is included in the attachment.

10.1 Check the motor protection switch

After installation of the compressor the motor protection switch needs to be checked and adjusted if it is not set up correctly. It was set to the recommended setting in the factory (see "4 Technical data").

- › Switch on the unit at the pressure switch by rotating the switch to the "I" position.

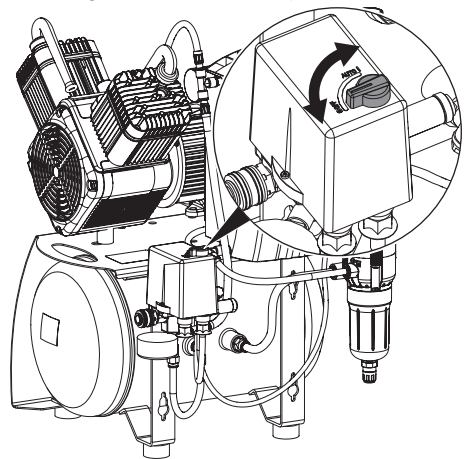


- › Measure the maximum current consumption (this is the value just before the cut-off pressure is reached).
- If the reading deviates from the recommended setting then the motor protection switch needs to be adjusted (see "11.2 Adjusting the motor protection switch").

10.2 Checking the switch-on/cut-off pressure

The switch-on/cut-off pressure is preset at the factory. Check the setting during first start-up.


- › Switch on the unit at the pressure switch by rotating it to the position "I AUTO".
 - › Read off the cut-off pressure from the pressure gauge.
 - › Drain the air from the pressure tank (e.g. via the condensate drain valve) until the unit starts and then close it again.
 - › Read off the pressure when the unit switches on.
- If the readings deviate from the values preset at the factory, adjust the pressure switch to the factory settings.

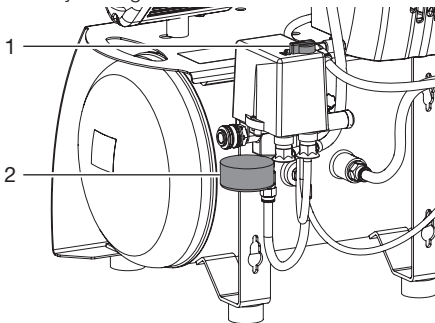

- 1 On/off switch
- 2 Pressure gauge

10.3 Checking the safety valve

Correct operation of the safety valve must be checked when the unit is started up for the first time and again subsequently at regular intervals.

-  At the factory, the safety valve is set to 10 bar (1 hPa), checked and stamped.



DANGER

Risk of explosion of the pressure tank and pressure hoses

- › Do not change the safety valve settings.

- › Switch on the unit at the pressure switch and fill the pressure tank to the cut-off pressure.




WARNING

Risk of damage to the safety valve

Risk of explosion of the pressure tank and pressure hoses due to a defective safety valve

- › Do not use the safety valve to vent the pressure tank.

- › To open, rotate the screw of the safety valve anti-clockwise until the valve begins to blow off. Only allow the safety valve to blow for a short period.
- › Then turn the screw clockwise as far as it will go to close the valve. The valve must now be closed again.

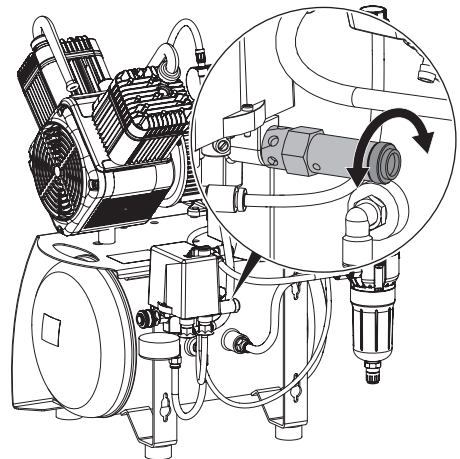



10.4 Draining the condensation water

Temperature changes during transport may cause condensation water to accumulate in the pressure tank. The condensation water can only be drained from the pressurised pressure tank.

- › Switch on the unit at the pressure switch and wait until the cut-off pressure is reached.

Pressure tank

- › At maximum tank pressure, slowly open the condensate drain valve.
- › Close the condensate drain valve as soon as all of the accumulated condensation water has been blown out.

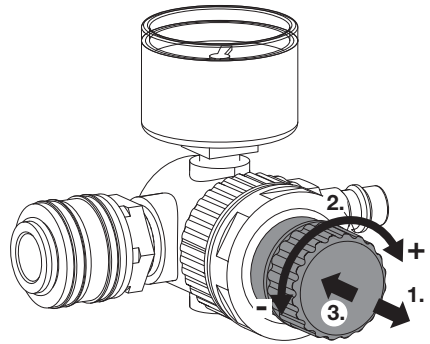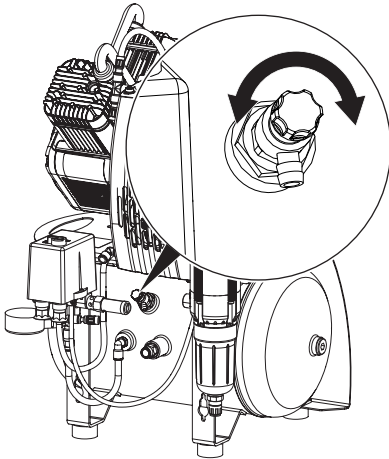

EN

10.5 Adjusting the rate of flow at the pressure reducer

The pressure reducer regulates the rate of flow in the system and adjusts it to the required operating pressure. In order to adjust the rate of flow air needs to be extracted via a consumer.

- › Activate the air consumer unit.
- › Lift the rotary knob at the pressure reducer.
- › Adjust the rate of flow via the rotary knob.
 - Turn the knob in the "+" direction to increase the rate of flow.
 - Turn the knob in the "-" direction to reduce the rate of flow.
- › Press in the rotary knob until it engages and cannot be adjusted.

11 Adjustment options

11.1 Adjusting the pressure switch

EN



WARNING

Risk of explosion of the pressure vessel

The pressure vessels used in the compressors are designed to withstand continuous pressure changes of 2 bar and can be used continuously under these pressure changes.

- › For load changes > 2 bar (max. permissible: 3 bar), comply with the maximum load change cycles specified in the operating instructions of the pressure vessel.



DANGER

Exposed live parts

Risk of electric shock due to live parts

- › Disconnect all power from the device.
- › Use insulated tools.
- › Do not touch live parts.



The cut-off pressure must be at least 0.5 bar (0.05 hPa) below the maximum pressure of 10 bar (1 hPa) of the safety valve. Otherwise the safety valve can open too early, which will prevent the compressor unit from attaining the cut-off pressure, as a result of which it will run continuously. The maximum permitted pressure is marked by a red line on the attached pressure gauge.


If the read-off values differ from the factory settings or if other settings are required, the cut-off pressure of the compressor can be adjusted at the adjusting screw on the pressure switch. The start-up pressure can then be adjusted using the pressure difference Δp .

- › Take off the pressure switch cover.
- › Adjust the cut-off pressure P at the adjustment screw.

The cut-off pressure increases in the "+" arrow direction and decreases in the "-" arrow direction. The pressure difference Δp is also influenced by this adjustment.

- › Adjust the start-up pressure via the pressure difference Δp at the adjustment screw. The pressure difference increases in the "+" arrow direction and decreases in the "-" arrow direction.


The maximum permissible pressure difference must not be set to more than 3 bar.

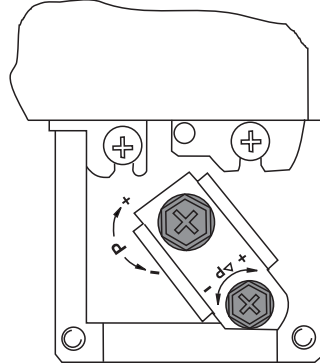

11.2 Adjusting the motor protection switch


Pressure switch

- › Take off the pressure switch cover.
- › Adjust the motor protection switch with the adjustment screw to the measured value (observe the range between the MIN permissible setting and the MAX permissible setting of the motor protection switch, see "4 Technical data").

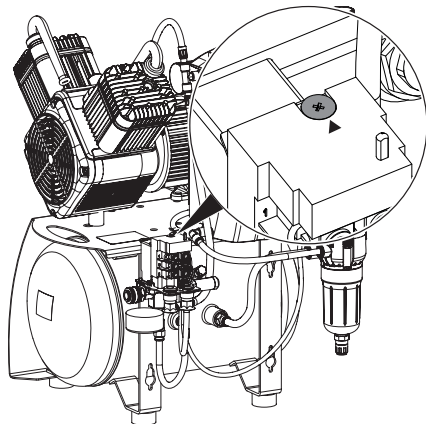

Controller


- › Remove the cover from the controller.
- › Adjust the motor protection switch with the adjustment screw to the measured value (observe the range between the MIN permissible setting and the MAX permissible setting of the motor protection switch, see "4 Technical data").

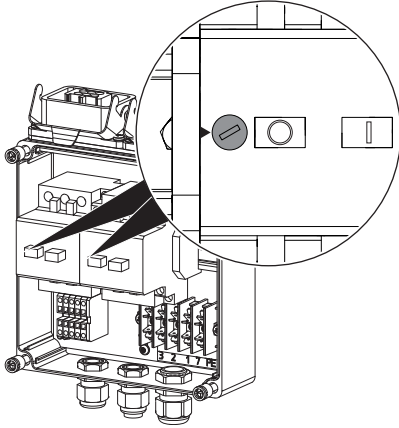

12 Circuit diagrams


12.1 1/N/PE AC 110-127 V, 230 V layout

EN

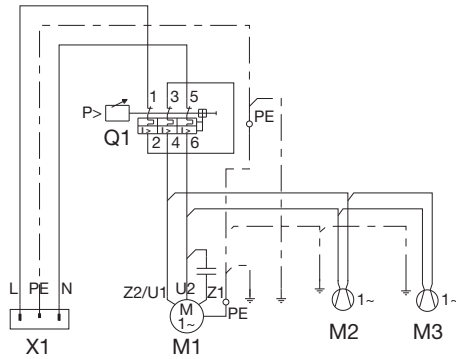


- X1 Mains connection L/N/PE AC 230 V
- Q1 Pressure switch
- M1 Compressor unit
- M2 Fan motor, membrane drying unit
- M3 Fan motor, noise insulation (if required)

12.2 3/N/PE AC 400 V layout

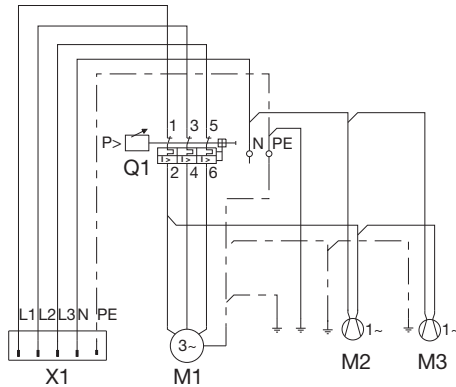

- X1 Mains connection 3/N/PE AC 400 V
- Q1 Pressure switch
- M1 Compressor unit
- M2 Fan motor, membrane drying unit
- M3 Fan motor, noise insulation (if required)


12.3 1/N/PE AC 230 V layout, Duo Tandem

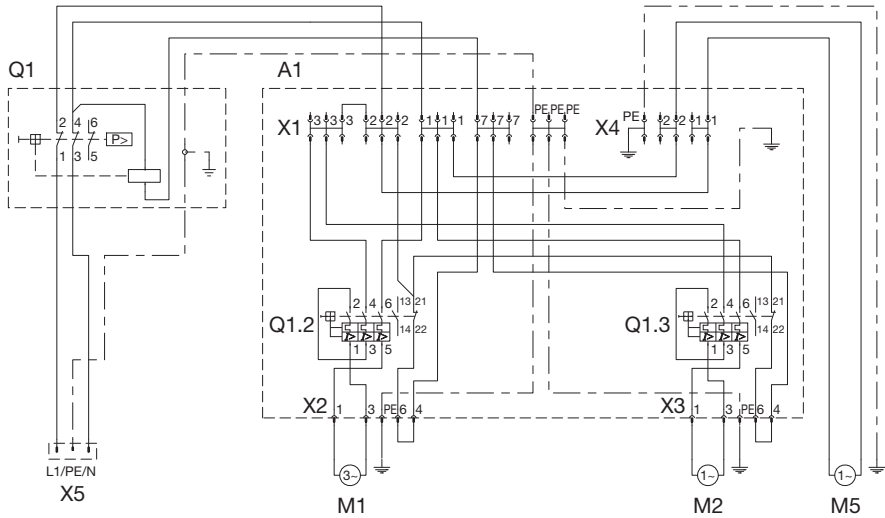

EN

- X5 Mains connection L/N/PE AC 230 V
- Q1 Pressure switch
- A1 Controller
- X1 Distributor rail
- X2 Plug connection of compressor unit
- X3 Plug connection of compressor unit
- X4 Distributor rail
- Q1.2 Motor protection switch
- Q1.3 Motor protection switch
- M1 Compressor unit
- M2 Compressor unit
- M5 Fan motor, membrane drying unit

12.4 3/N/PE AC 230 V layout, Quattro Tandem

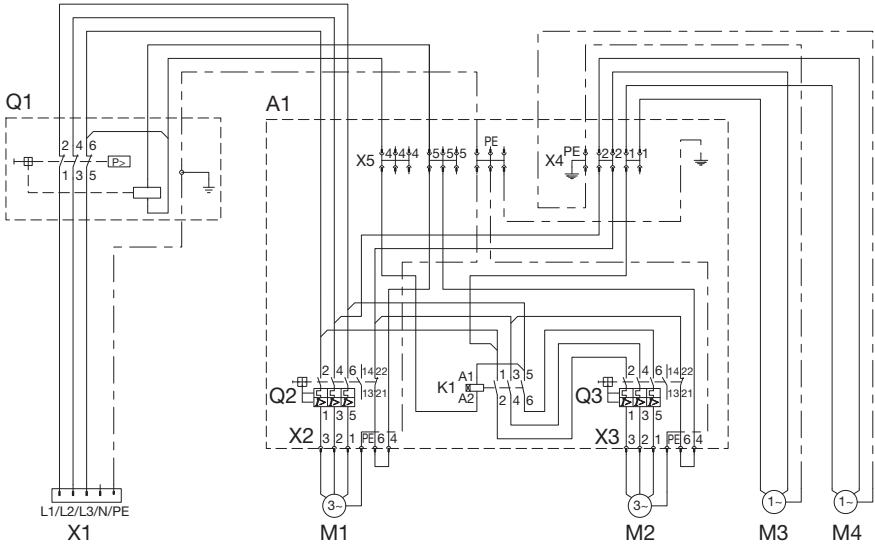

- X1 Mains connection 3/N/PE AC 230 V
- Q1 Pressure switch
- A1 Control box
- X2 Plug connection of compressor unit
- X3 Plug connection of compressor unit
- X4 Distributor rail
- X5 Distributor rail
- Q2 Motor protection switch
- Q3 Motor protection switch
- K1 Time-lag relay
- M1 Compressor unit
- M2 Compressor unit
- M3 Fan motor, membrane drying unit
- M4 Fan motor, membrane drying unit


13 Operation



Prior to working on the unit or in case of danger, disconnect it from the mains.

13.1 Switching the unit on/off

- › Switch on the unit at the pressure switch by rotating it to the position "I AUTO".
The compressor unit will start up automatically and fill the pressure tank. When the cut-off pressure is reached the compressor unit switches itself off automatically.
- › The unit can be switched off when required by turning the pressure switch to the "0 OFF" setting.

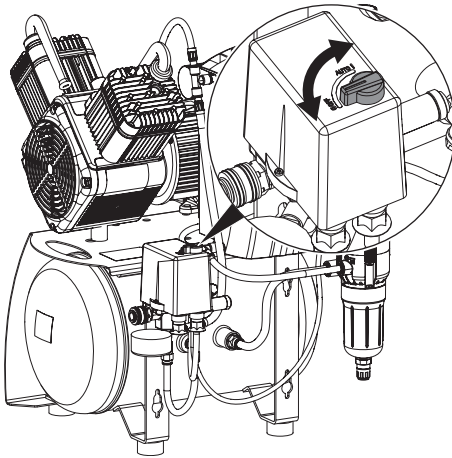

14 Maintenance

EN



Prior to working on the unit or in case of danger, disconnect it from the mains.



VORSICHT

Risk of infection due to burst filters

Particles enter the compressed air network and can therefore enter the mouth of the patient.

- › Replace filters in accordance with the maintenance schedule.

14.1 Maintenance schedule



ACHTUNG

Risk of damage to the unit due to blocked filters


Continuous running due to reduced delivery. Damage to the unit due to burst filters.

- › Replace filters in accordance with the maintenance schedule.

Maintenance interval	Maintenance work
At regular intervals	› Empty the collector tray under the membrane drying unit (the interval may vary depending on the ambient conditions and method of working; empty it daily if the humidity is high).
Annually	› Replace the air intake filter in the compressor unit – every six months if the concentration of dust is high. › Replace the fine or sterile filter. › Replace the sintered filter.
Every 4 years	› Replace the vibration dampers.
In accordance with national law	› Check the safety valve. › Carry out recurring safety inspections (e.g. pressure tank inspections, electrical safety inspections) in accordance with applicable national laws.

14.2 Replacing the air intake filter

- › Switch off the compressor at the pressure switch.
- › Pull off the noise reducer from the air intake filter.
- › Remove the air intake filter.
- › Insert a new air intake filter.
- › Push on the noise reducer onto the air intake filter.

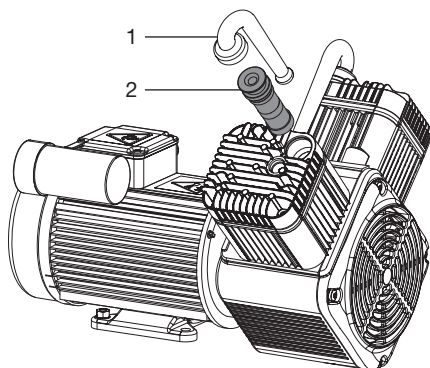


- 1 Noise reducer
- 2 Air intake filter

14.3 Replacing the filter of the membrane drying unit


Fine/sterile filter

- › Switch off the unit.
- › Disconnect all power from the device.
- › Unscrew and remove the filter cover.
- › Remove the fine/sterile filter.
- › Insert the new fine/sterile filter.
- › Replace the filter cover and close.

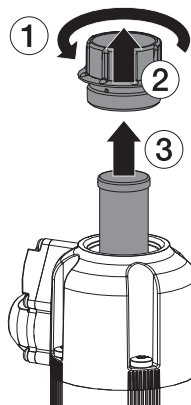

Sintered filter

- › Unscrew and remove the filter housing.
- › Remove the sintered filter.
- › Insert a new sintered filter.
- › Replace the filter housing and close.

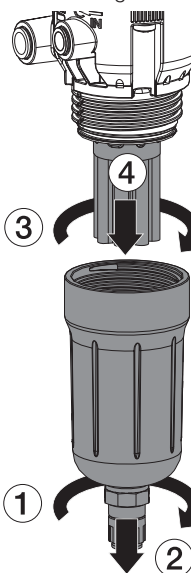

15 Taking out of use

15.1 Taking the unit out of use



If the unit is not to be used for a prolonged period of time, we recommend that it is properly shut down and taken out of use.

To do this, the accumulated condensation water from the unit must be drained.

- › Switch on the unit and wait until the cut-off pressure is reached.

Membrane drying unit

- › While the compressor unit is running, open the condensate drain valve on the membrane drying unit. When no more condensation water emerges, close the condensate drain valve.
- › Switch off the device.

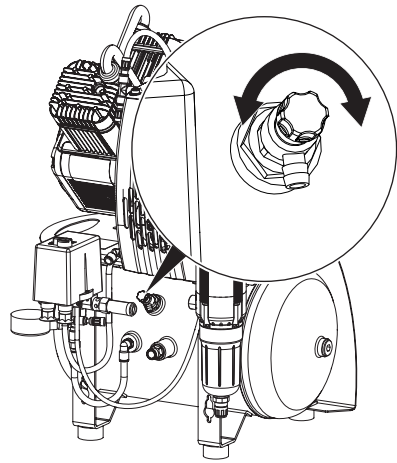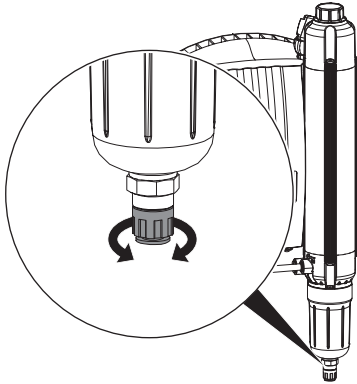

15.2 Storage of the unit



WARNING

Risk of explosion of the pressure tank and pressure hoses

- › The pressure tank and the pressure hoses must be vented before they are stored or transported.
- › Protect the unit against moisture, dirt and extreme temperatures during transport (refer to the section on "Ambient conditions").
- › Only store the unit when it has been completely emptied.


Pressure tank

- › Open the condensate drain valve. Once the start-up pressure has been reached the compressor will switch on.
- › With the compressor switched on and the condensate drain valve open, wait until no more condensation water emerges.
- › Switch off the unit.
- › Close the condensate drain valve when no more air escapes.
- › Disconnect all power from the device.
- › Disconnect the compressed air connection on the quick release coupling.

? Troubleshooting

EN

16 Tips for operators and service technicians

 Any repairs exceeding routine maintenance may only be carried out by qualified personnel or our service.

 Prior to working on the unit or in case of danger, disconnect it from the mains.

Error	Possible cause	Remedy
Compressor will not start	No mains voltage. On three-phase units: one phase is missing or not connected (generation of a humming sound)	<ul style="list-style-type: none"> › Check the mains fuse; if necessary, switch the circuit breaker back on. If the fuse is defective, replace it. Check the mains supply voltage. ›
	Undervoltage or overvoltage	› Measure the supply voltage; call an electrician if necessary.
	Relief valve defective, unit starts against pressure	› Check that the relief valve discharges after switching off the unit. Free up the movement of the relief valve operable or replace it.
	Mechanical sluggishness of a unit (piston is stuck); motor protection has tripped	› Switch the unit off and disconnect it from the power supply, remove the fan hood from the blocked compressor and turn the fan wheel. If this is not possible, replace the piston and cylinder or the complete unit.
Humming noise from motor	Motor capacitor is defective	› Replace the capacitor.
Compressor does not switch off	Wrong size of compressor, air intake too high	› Calculate the air requirement (this can be up to 50 l/min per treatment unit), if necessary install a larger compressor.
	Leak in the compressed air system	<ul style="list-style-type: none"> › Locate and seal the leak. › Inform a service technician.
	Defective membrane drying unit	› Check whether there is an increased flow of air at the filter housing of the membrane drying unit (bottom), if necessary replace the membrane drying unit.

Error	Possible cause	Remedy
Compressor switches on from time to time even though no air is being taken for a consumer unit	Leak in the compressed air system	<ul style="list-style-type: none"> ➤ Locate and seal the leak. ➤ Inform a service technician.
Knocking or loud noises on the compressor	Compressor unit defective	<ul style="list-style-type: none"> ➤ Disconnect all power from the device and inform a service technician.
Reduced delivery. Compressor needs longer to charge the pressure tank, see charging times in "4 Technical data"	Air intake filter dirty	<ul style="list-style-type: none"> ➤ Replace the air intake filter at least 1x per year. The air intake filter must never be cleaned.
	Defective membrane drying unit	<ul style="list-style-type: none"> ➤ Replace the membrane drying unit. ➤ Inform a Service Technician.
Water dripping from air consumers	Maintenance work not carried out regularly (without membrane drying unit)	<ul style="list-style-type: none"> ➤ Regularly drain the condensation water from the pressure tank, see "10.4 Draining the condensation water"
	Defective membrane drying unit	<ul style="list-style-type: none"> ➤ Inform a service technician.

 Appendix

EN

17 Handover protocol

This document confirms the qualified handover and provision of instructions for the medical device from Dürr Dental. This must be carried out by a qualified adviser for the medical device, who will instruct you in the proper handling and operation of the medical device.

Product name	Order number (REF)	Serial number (SN)

- Visual inspection of the packaging for any damage
- Unpacking the medical device and checking for damage
- Confirmation of the completeness of the delivery
- Instruction in the proper handling and operation of the medical device based on the operating instructions

Notes:

Name of person receiving instruction:**Signature:**

Name and address of the qualified adviser for the medical device:

Date of handover:**Signature of the qualified adviser for the medical device:**

--	--



Hersteller/Manufacturer:

DÜRR DENTAL SE
Höfigheimer Str. 17
74321 Bietigheim-Bissingen
Germany
Fon: +49 7142 705-0
www.duerrdental.com
info@duerrdental.com

